DEPANAREA

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EN

Dear clients,

Thank you for choosing our CANGAROO nose cleaner AKIKO. We hope that it will be useful for you.

IMPORTANT! READ CAREFULLY AND KEEP FOR FUTURE REFERENCE! The product is tested according and it responds to the requirements of the following European standards: EN 55014-1, EN 55014-2; EN 61000-3-2 и EN 61000-3-3.

RULES AND RECOMMENDATIONS FOR SAFETY USAGE

1. Use the product only as intended. The device must be used only for cleaning of the baby's/child's nose from secretions and mucus. Do not use the product for other purposes.
2. **Attention!** This product is not a toy. Children must not play with the device.
3. Only adult can work with the device. Keep away from children's access.
4. Cleaning and maintenance of the device must not be done by children.
5. **Attention!** This device is not intended for using by persons (including children) with limited physical, sensory or mental abilities, or people with insufficient experience or knowledge, unless they have been observed or structured, about the safe using of the device and they understand the dangers.
6. Use only accessories and spare parts, provided by the manufacturer.
7. Do not open, do not drop and do not throw the device, otherwise the will be damaged.
8. Use only alkaline batteries. Do not use rechargeable batteries for this product.
9. Take out the batteries, when the product will not be used for a long time, to avoid leaking and corrosion.
10. Keep the manual on easy accessible for all users place but away from children.
11. Remember that the working mode of the product is with interruptions. The recommended time for usage is 5 minutes, after which the device should rest at least 10 minutes.
12. **The manufacturer keeps the right to change the manual and specifications for the product without pre-warning!**
13. Colors and specifications of the product, which you have bought can differ from those, described in the manual.

IMPORTANT BATTERY GUIDELINES

1. An adult must work with batteries.
2. Batteries are not toys! Children must not play with batteries. Swallowing hazard! Small children can swallow the batteries and to suffocate. Because of this, storage the batteries on non-accessible for children place!

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3. Always remove the exhausted batteries and do not throw them together with the household waste but on the intended places. They are recyclable.
 4. Use only AA size batteries.
 5. Alkaline batteries are recommended.
 6. When you replace exhausted batteries with new, always change all batteries.
 7. Always place batteries correctly according their polarity (+/-), marked on the bottom of battery compartment. If they are placed incorrectly, this can cause short circuit or electric shock.
 8. Do not mix old and new batteries.
 9. Do not mix alkaline, standard (Carbon - Zinc) or rechargeable (Nickel - Cadmium) batteries.
 10. Do not put the batteries in fire, because they can explode or to leak.
 11. Keep the batteries away from direct heating sources.
 12. If you will not use the device for long time, take the batteries from the battery compartment.
- This is how you will protect the device from battery leaking and corrosion.
13. If when changing the batteries, you find that the batteries are leaking, put protective gloves and clean the battery compartment with dry cloth.
 14. Non-rechargeable batteries must not be recharged.
 15. Do not disassemble, open or shatter the batteries.

NOTES BEFORE USE

1. Do not forget to sterilize the accessories before using, including the silicone suction nozzle and the transparent cover.
2. Attention! The device itself must not be washed under running water, neither to get sterilize!
3. Wash your hands before using the product and accessories.
4. Check silicone seal ring.
5. Do not put the device in liquids.

PRODUCT CHARACTERISTICS

1. Two adjusting the suction power options.
2. With musical function (option)
3. Comfortable hygienic caps for secretion.
4. Keeps baby's nasal cavity clean.
5. Design, imitating shape of dolphin.

The nose cleaner is a device, which helps for innovative, pure and effective care for the health of your baby. It is brand new product, easy for maintenance.

ADVICE TO PARENTS

1. Wash your hands well before to care of your child.
2. Lift the child's head by the time of using the nose cleaner and ensure that the air flow is moderate.
3. Ensure that the baby's mouth is open by the time of using the product, so the air can go into the nostrils.
4. Avoid infectious sources:
 - A. Do not kiss sick baby.
 - B. Do not kiss the baby, when you are sick.
5. Isolate sick babies.

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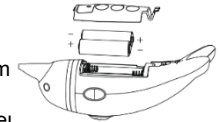
5. If a problem with the device occur, contact with the trade person, which you bought the product from or with authorized repair shop for consultation. Do not try to repair the product by yourself, because this way your warranty will be cancel.

PRODUCT DESCRIPTION

Look page 2.

A. Air inlet; B. Silicone seal ring; C. Basis of plastic cup; D. Button for suction level/ turning off; E. Turning on/off button for musical functions; F. Battery compartment cap; G. Loudspeaker; H. Plastic cap for secretion; I. Silicone suction nozzle; J. Transparent cover

TECHNICAL CHARACTERISTICS

- 1.Environment temperature: 10°C - 40 °C
Relative humidity: ≤80% RH
2. Batteries: (2 pcs. AA batteries)
- 3.Product size: 190x60x54 mm (L x W x H)
4. Product weight (with batteries): 196 g
5. Maximum suction power: 58 kPa

PRODUCT ASSEMBLING

Look page 2.

1. Open the cap of the battery compartment (1).
 - 2.Place 2 batteries AA 1.5V size correctly, according the polarity, marked on the bottom of the compartment.
 3. Do not use rechargeable batteries and do not mix old with new batteries.
 4. Put back the cap of the battery compartment.
 5. After putting the batteries back (while using for the first time), a hygienic cap (4) must be placed in the sealing base (3).
- A.Ensure that the silicone seal (2) is correctly places, so it can make good vacuum.
B. Put the transparent cap (5) on the correct place and turn it, until it get fixed.
C. Place the silicone suction nozzle (6).

INSTRUCTIONS FOR USE

1. Place the batteries and assemble the product, as described.
 2. press the button on level "I" or on level "II" .
 - 3.Place the front part on the silicone nozzle in the baby's nasal cavity carefully (please, hold baby's head in straight position), then clean the nasal cavity.
 4. After you stop using the product, put the button for level/ turning off in position "O".
 5. By the time of usage, please, press the musical button in position „🎵“, than different melodies will sound. When the button is on „O“, the melodies will stop.
- Ensure that baby's mouth is opened by the time of using the product and that there is air flow in the nasal cavity.
- Do not suck secretion from the nose wall.
 - Low the movement, it must not be rough and suddenly, to not harm the baby.
 - Change the hygienic cap if it is necessary.
- **Attention!** When you clean your child's nose with saline, it can flow to bronchi, mixed with the nasal secretion. Ensure that you have sucked this mixture fully from the nose.

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- **Attention!** Ensure that the child's mouth is opened by the time of using the nose cleaner, so that air flow can occur.
- **Attention!** Throw the hygienic cap, when it is already necessary and clean the device in proper way.
- **Attention!** Do not immerse the device in water or other liquids.
- **Attention!** Sterilize movable parts after usage.

TROUBLESHOOTING

If some of the following problems occur, please make a reference with this point of the manual. If the problem is still on, contact with the trade person, which you have bought the product from.

Problems	Solutions
No aspiration power	1. Check the sealing connection. 2. Check if the hygienic cover is on its place or not. 3. Check whether the silicone nozzle is on its place or not.
No reaction with switch ON/OFF button	Check whether you have place the batteries correctly according their polarity.

MAINTENANCE

1. Wipe the main part of the device with dry soft cloth. Do not pour it over with water or other liquids, to prevent damage due to short circuit.
2. Take off the movable parts – the silicone nozzle, the silicone seal ring and the transparent cover and sterilize them with boiled water or steam, but not in microwave oven.
3. Do not expose the product on direct sunlight and humidity.
4. Do not use corrosive chemicals for cleaning the device and strong detergents – those on alcohol basis, bleach or with abrasive particles.
5. Storage the product on dry and clean place, away from direct sunlight.
6. Take the batteries out, when the device will not be used for long period of time. This is how you will avoid their leaking or corrosion.

ACCESSORIES, INCLUDED IN THE SET

- 2 pcs AA alkaline batteries;
- 1 pcs User manuals;

15 pcs. Plastic hygienic caps

1pcs. Silicone suction nozzle

1 pcs . Silicone seal ring

Produced for CANGAROO in China
Importer: Moni Trade Ltd.

Address: Bulgaria, Sofia, Trebich quarter, ul. Dolo 1

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WICHTIG: MONI TRADE LTD