BG: ИНСТРУКЦИЯ ЗА УПОТРЕБА
EN: USER MANUAL
GR: ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
RO: INSTRUCȚIUNI DE UTILIZARE
DE:GEBRAUCHSANWEISUNG
HU: HASZNÁLATI ÚTMUTATÓ

ES: INSTRUCCIONES DE USO
IT: ISTRUZIONI PER L'USO
РУ: ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ
SR: UPUTSTVO ZA UPOTREBU
NL: GEBRUIKERSHANDLEIDING
FR: NOTICE D'UTILISATION

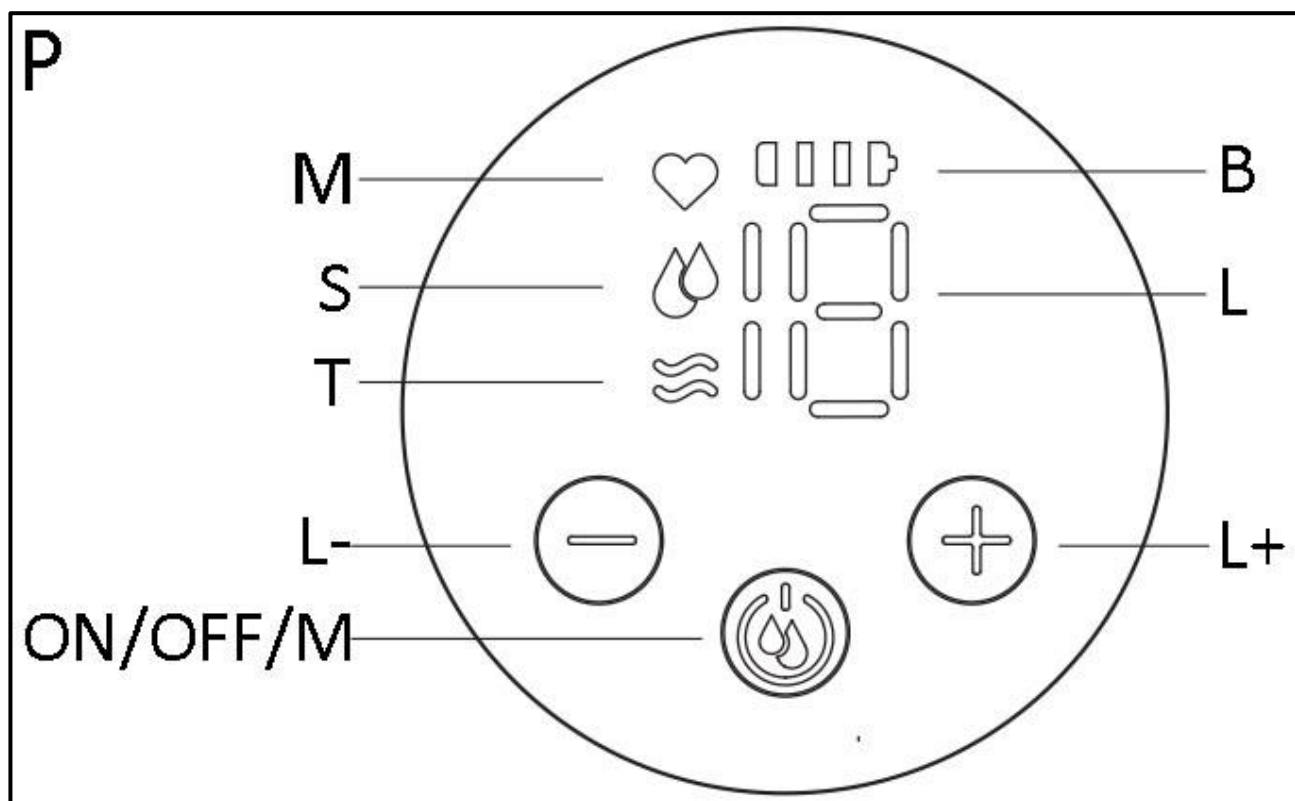
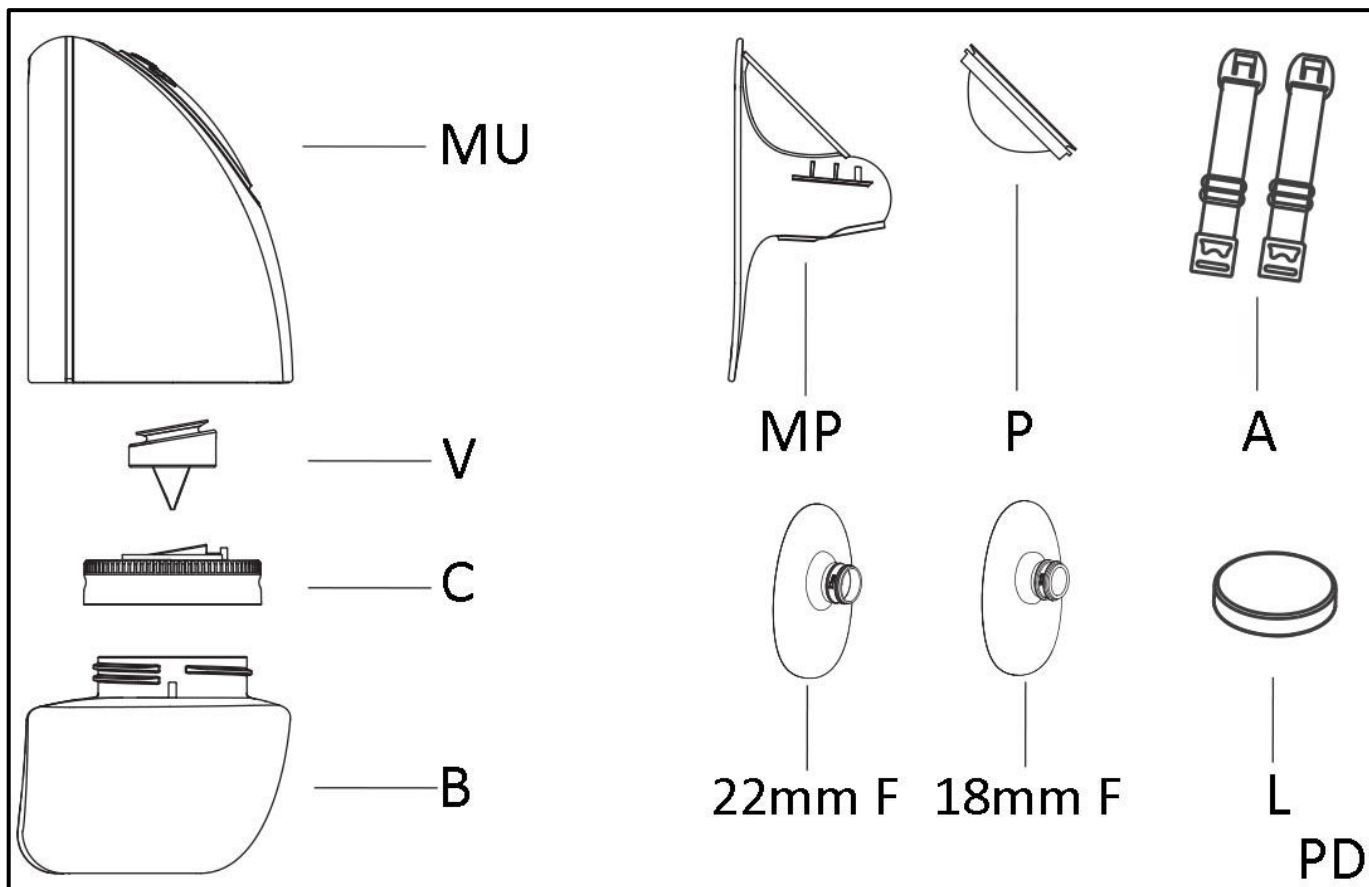
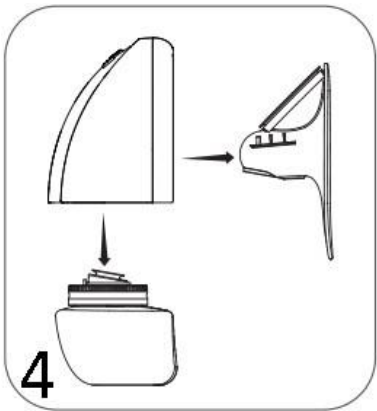
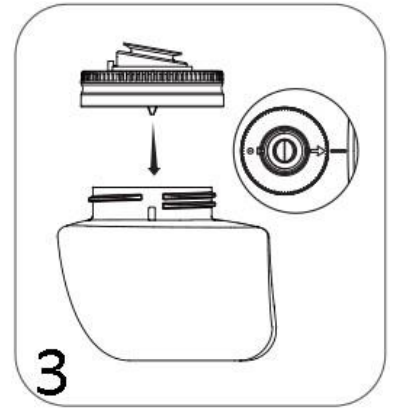
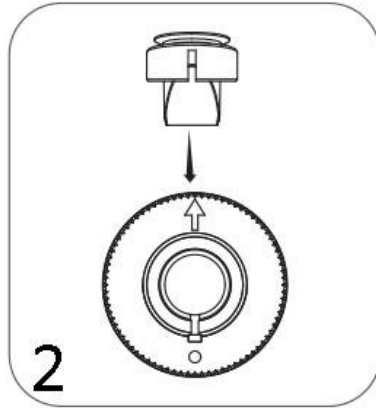
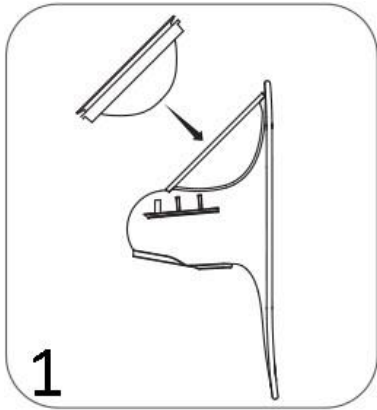
LIBERA

XN-D268




BG: ЕЛЕКТРИЧЕСКА ПОМПА ЗА КЪРМА
EN: ELECTRIC BREAST MILK PUMP
GR: ΗΛΕΚΤΡΙΚΟ ΘΗΛΑΣΤΡΟ
RO: POMPĂ ELECTRICĂ DE SÂN
DE:ELEKTRISCHE MILCHPUMPE
ES: EXTRACTOR DE LECHE MATERNA ELÉCTRICO
IT: TIRALATTE ELETTRICO
РУ: ЭЛЕКТРИЧЕСКИЙ МОЛОКООТСОС
SR: ELEKTRIČKA PUMPA ZA MAJČINO MLEKO
NL: ELEKTRISCHE BORSTKOLF
FR: TIRE-LAIT ÉLECTRIQUE
HU: ELEKTROMOS ANYATEJ SZIVATTYÚ

5

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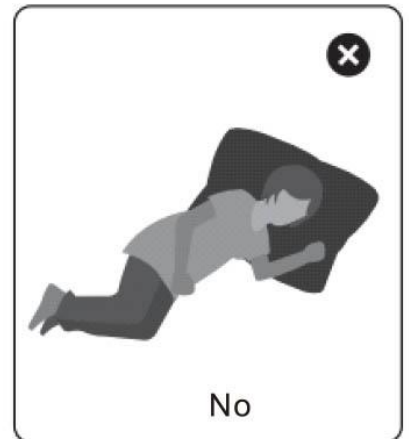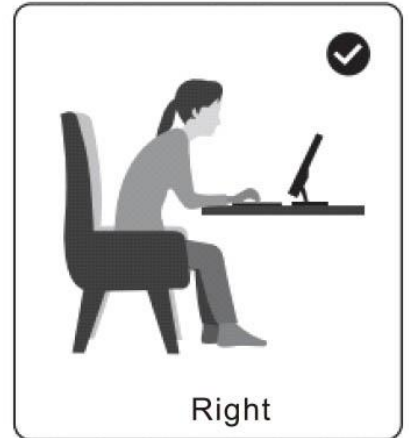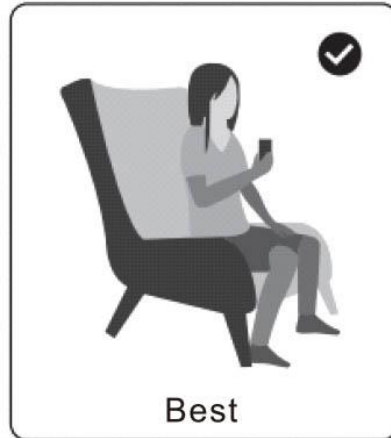IMPORTANT! READ CAREFULLY AND KEEP FOR FUTURE REFERENCE!

THESE INSTRUCTIONS CONTAIN IMPORTANT INFORMATION, GUIDELINES AND RECOMMENDATIONS FOR THE PRODUCT AND ITS SAFE USAGE.

Congratulations on the purchase of this electric pump. This product is an electric breast pump uniquely designed with different levels of control of the suction power and unique valve with gentle suction.

➤ **The Cangaroo electric breast pump provides many of the features which the busy mothers love and is perfect for many reasons:**

- Expressing and collecting breast milk while you are away from your baby.
- Relieving the tension in your breasts caused by the large amount of breast milk. For example, when your baby's feeding habits change because it has started eating more solids.
- Maintaining your milk supply if you have to temporarily stop breastfeeding.

➤ **The Cangaroo electric breast pump has the following features:**

1. Massage mode – 5 Levels.
2. Suction mode – 10 levels.
3. Combination mode massage with suction – with 10 levels.
4. Recovery mode – 3 Levels, intended for women with short, inverted or flat nipples.
5. Quiet motor providing discretion.
6. Light and easy to carry.

I. RECOMMENDATIONS AND WARNINGS FOR THE SAFE USAGE OF THE PRODUCT

1. Use this product only for its intended purpose as described in this manual
2. This unit is not intended for usage by persons (including children) with reduced physical, sensory or mental capabilities or the lack of experience and knowledge unless they have been given supervision or instruction by a person responsible for their safety.
3. **Warning!** The breast pump is intended for repeated use only by one user.
4. **Warning!** Children should not play with the manual breast pump.
5. **Warning!** Before first usage check the breast pump, including the adaptor for signs of damage. If you find such, do not use the product until the fault is removed. Do not try to modify or repair the product by yourself. For consultation contact an authorised service or the sales agent, from which you have bought the manual breast pump.
6. Do not use any accessories, which are not approved by the manufacturer and do not make any improvements or changes on the product as this may lead to its malfunction or injuries to the person, who uses it.
7. Do not use spare parts, which are not the original ones supplied by the manufacturer or the importer. Using such, Your warranty card will be invalid.
8. **WARNING: To reduce the risk of electrocution, please:**
 - Do not use while bathing or showering.
 - Do not place or store this product where it can fall or be pushed into a bathtub or sink.
 - Do not place or drop into water or other liquid.
 - Do not reach after a product that has fallen into water. Unplug from the wall socket immediately.
 - Never operate this product if it has a damaged adapter or plug, if it is not working properly, if it has been dropped or damaged, or if it has been dropped into water.
 - Make sure the voltage of the AC adaptor is compatible with the power source.
 - Only use the AC adaptor which is included in the set. If any damage of the adaptor occurs, contact the manufacturer or the sales agent, from whom you have purchased the product.

- The product should not be left unattended when plugged into the electrical wall socket.
 - Do not approach high temperatures or fire sources during the use of the pump.
9. Do not use while you are pregnant as this may lead to stimulating the birth process.
 - 10.Never use while you are sleepy or drowsing.
 - 11.Do not leave it and do not keep the product in a place with too high room temperatures or at direct sunlight.
 - 12.Be careful not to drop the product and to put in its opening or in the feeding bottle other objects or liquids.
 - 13.If you are infected with Hepatitis A, Hepatitis B or HIV (Human Immunodeficiency Virus), pumping breast milk does not reduce or remove the risk of transmitting the virus to your baby through breast milk.
 - 14.Do not attempt to remove the pump body from your breast while under vacuum. Turn the unit off and break the seal between your breast and the silicone pad with your finger, then remove the pump from your breast.
 15. Wash, rinse and sterilize all the parts of the breast pump after each use. Do not clean the part of the breast pump with strong detergents – alcohol-based ones, such containing bleach or with abrasive parts.
 - 16.Never wash and sterilize the electrical parts of the product – the motor of the pump and the AC adaptor in order not to cause permanent damage to the product.
 - 17.WARNING! This product is intended for personal usage, and it is not intended for the usage of other mothers.
 - 18.The breastmilk bottle in the set is an integral part of the breast pump. For the purposes of this product use only this bottle and only for the breast milk collected with this pump.

II. PARTS DESCRIPTION AND CONTROL PANEL USE OF ELECTRIC PUMP

- Figure PD: MU – Pumping motor (mechanism) – 1 pc; V – Silicone valve – 2 pcs; C – Milk container cover – 1 pc; B – Milk container (180ml/ 6.00 oz) – 1 pc; MP – Massage pad – 1 pc; P – Silicone piston – 1 pc; A – Bra adjuster – 1 set (left and right); 22mm F – 22 mm Silicone insert – 1 pc; 18mm F – 18 mm Silicone insert – 1 pc; L – Lid for the milk container – 1 pc.
- Figure P – **ON/OFF/ M**: Button ON/ OFF/ MODE; **L-**: Level down; **T**: Recover mode; **S**: Suction mode; **M**: Massage mode; **B**: Battery level; **L**: Level indicator; **L+**: Level up;

III. CHARACTERISTICS

Type	Parameters
Battery capacity	3.7V -1300mAh
Rated input voltage	5V
Rated Input Current	1A
Working Power	5W
Charging time	120 minutes
Usage time	150 minutes
Display	LED
Method of control	Buttons

Type	Parameters
Material used to manufacture the bottle cap	PP
Material used to manufacture the valve and piston	Silicone
Material used to manufacture the milk container, massage pad	PCTG

IV. BATTERY CHARGE

1. Before first time use charge the battery completely.
2. The breast pump is with a built-in battery 3.7V/ 1300 mAh.

3. Charging:

- Connect the adapter to the charging jack of the pumping mechanism.
- Insert the plug into electrical outlet of the mains supply of your home.
- We recommend you to leave the breast pump to charge completely before next use in order to prolong its battery life.

4. Use only the charger provided with the breast pump. Do not use chargers from other products.

5. Charge the battery on time. Charge as soon as the battery indicator is with 1 bar or the indicator starts flashing.

6. During charging, the battery indicator will blink. Once the battery is fully charged, the indicator will stop flashing.

7. If you will not use the product for a long time, charge the battery completely before storing it. Then charge the battery at least once a month to prolong its life.

8. If the battery is damaged or cannot be charged anymore, do not try to open the product by yourselves. For consultation and repair contact the shop from where you bought the product or the importer/ distributor.

9. The battery is recyclable. If the product is damaged and cannot be used any longer, take it to an official collection point for electronic products' waste, batteries, chargers. DO NOT throw the product away with the household waste. Follow your country's rules for the separate collection of electrical and electronic products and rechargeable batteries. Dispose of waste of this type only at the designated places. This will be important and significant contribution for the environmental protection and the reduction of the harmful effects of the substances contained in batteries and electronic components on the environment and human health.

V. ASSEMBLY OF THE ELECTRIC BREAST PUMP

ATTENTION! NEVER STERILIZE, CLEAN WITH WATER OR SUBMERGE THE PUMP MECHANISM AND CHARGER IN WATER!

Follow the product assembly figures to ensure proper assembly.

1. Step 1 (Figure 1): Attach the piston to the upper part of the massage pad. The ends of the piston must fit tightly to the ends of the massage pad. The shape of the piston must fit perfectly to the shape of the massage pad.

2. Step 2 (Figure 2): Attach the silicone valve to the milk container cap (C). Make sure that it is attached in the correct position and the opening of the valve matches the support point of the container cap. **Important!** If the valve is loose or is not attached properly, it will be hard to make vacuum.

3. Step 3 (Figure 3): Attach the cap (C) to the milk container and twist it clockwise. The arrow on the cap must be aligned with the sign on the milk container.

4. Step 4 (Figure 4): Attach the massage pad with the piston to the pumping mechanism. Make sure they fit completely tightly to each other. Then attach the milk container to the bottom part of the pumping mechanism. They should fit tightly to each other and there not to be any space between them.

5. Step 5: With clean hands attach your size silicone insert to the massage pad. It should fit tightly to the pad.

VI. USE

• Wash your hands thoroughly and make sure your breasts are clean. Express some milk to check that your milk ducts are not blocked. The pump must be sterile and assembled.

• Sit down and relax in a comfortable chair, leaning forward slightly (Use a soft and comfortable pillow as a backrest).

1. Placing the breast pump: Put the pump on your breast. The silicone breast (nipple) shield should be firmly fitted against the breast, and the nipple should be centrally directed towards the breast pump opening. Keep the pump upright.

Be careful not to move the pump because the lost of good contact with the breast will cause bad vacuum. Put your bra back over the pumping mechanism. Figure shows the correct and incorrect positions to stand in while using the pump.

2. Use:

A. Massage mode (M):

Press the ON/ OFF/ MODE button for a several seconds until the pump switches on. Short press the button again to choose the mode. By default the massage mode will start working. A heart-shaped icon will appear on the panel indicating that the massage mode is on. This mode imitates the natural way of sucking of the baby in the beginning of breast feeding, which is fast and light. Thus, the reflex for flow (secretion) of breast milk is stimulated and initiated. This mode is with 5 levels. Using the + (L+) and – (L-) buttons you can regulate the massage levels. After 2 minutes of massage the mechanism will automatically switch to suction mode.

B. Suction mode (S):

Short press the button ON/ OFF/ MODE to choose the suction mode. An icon with the shape of 2 drops will appear on the panel. The mode mimics a baby's natural sucking with a high and gentle suction force to achieve an efficient result. It has 10 levels of suction and using the + (L+) and – (L-) buttons you can select your preferred level.

C. Combined mode:

Short press the button ON/ OFF/ MODE to choose the combined mode. Two icons will appear on the panel – one heart-shaped and one with the form of drops. The mode combines massage (stimulation) with suction and is with 10 levels. Using the + (L+) and – (L-) buttons you can select your preferred level.

D. Recover mode:

This mode is suitable for women with short, flat or inverted nipples. With its help, inverted nipples can be pulled out and the pumping process can be improved for women with the problems mentioned above. Press the ON/OFF/M button briefly to select recovery mode. 2 wave icons will appear on the panel. This mode has 3 levels. Using the + (L+) and – (L-) buttons, you can select your preferred level.

3. Memory function: Turn off the pump mechanism and the last used levels in massage mode and in suction mode will be automatically memorized.

➤ **Note:** The display will switch off automatically 20 seconds after selecting the preferred mode. This is to save energy. The pump will continue to run. Pressing any button will turn the display back on.

4. Once you have finished pumping, remove carefully the pump from your breast. Press the button on the front of the pumping mechanism and pull down the milk container. Transfer the breast milk into a feeding bottle or milk storage bag, or close the container using the lid (part L).

5. Tips:

- If there is no result during pumping, do not continue! Try using the pump at another time of the day.
- On average, you have to pump for 8 minutes to drain 120 ml of milk. Keep in mind that the amount of milk varies from woman to woman.
- Do not overfill the milk bottle to prevent milk from leaking out of the bottle.

VII. WHEN TO USE YOUR BREAST PUMP

- 1. If it is possible.** There should be a period of breastmilk secretion and breastfeeding before each use of the pump.
2. The electric pump is an apparatus, which helps postpartum women to ease the pain and to extend the breastfeeding time. It uses the premise that there is sufficient milk and smooth mammary gland. For some women, which are mothers for the first time, the mammary gland usually is not smooth enough. A reason sometimes can be the congenital nipple inversion or milk water shortage. So, we suggest you before purchasing and using the pump, to consult with a doctor who could confirm that your mammary gland is smooth and that the breast milk is in normal condition. Do not buy and use the pump only as a device to unblock your milk ducts.

VIII. HINTS

It takes practice in order to use the breast pump the best possible way. So it will take you at least several attempts before starting successfully to use the electric breast pump.

1. Choose a quiet time to use the pump – time, when no one will disturb you.

2. Sit in a comfortable chair, in a perpendicular position to the ground. Please, put a comfortable cushion behind your back before you start using the breast pump. Take a glass of water and place it near you, because you may feel thirsty during the pumping process.
3. Place a warm cloth on your breasts for a few minutes before starting to express the milk as this may facilitate the process and relieve the tension in the breasts.
4. Warmth and relaxation can help for the easier pumping of the breast milk, so before using the pump, you might like to take a bath or a shower.
5. Try expressing from one breast while your baby is feeding from the other, or continue expressing just after the feed.

IX. HOW TO STORE THE BREAST MILK

1. Only breast milk that has been collected using a sterile breast pump can be stored to feed your baby.
2. The milk can be stored in a thermos at 40 ° C, so you can feed your baby every 4 hours.
3. If you store freshly expressed milk, seal the milk. The milk can be stored for 4 hours at room temperature at 19 - 25 ° C.
4. The expressed milk can be stored in a refrigerator for a maximum of 48 hours (but not on the refrigerator door).
5. The frozen breast milk (at temperature -18°C) can be stored up to 6 months. When deep freezing at -20°C, the breast milk can be stored for up to 12 months.
6. If there is some milk left in the bottle after feeding the baby, you can use this milk within a maximum of 2 hours.
7. Frozen milk must first be left in the refrigerator to thaw naturally. If you need it urgently, put it in a bowl of warm water and thaw it. After thawing, the milk can be stored in the refrigerator for 24 hours. **NEVER FREEZE MILK AGAIN.**
8. Frozen milk can be gradually warmed in warm water or placed in a baby bottle warmer.
9. **NEVER WARM YOUR BREAST IN THE MICROWAVE, BECAUSE YOU CAN DESTROY VITAL NUTRIENTS AND ANTIBODIES THIS WAY.** Also, uneven heating can lead to too hot areas of breast milk, which can cause burns to your baby's upper digestive tract.
10. Never place a frozen bottle of milk in boiling water, as this may cause the bottle to burst / break. Never pour boiling water directly into a feeding bottle. You should allow the water to cool for about 20 minutes before filling the bottle.

X. CLEANING AND STERILIZATION OF THE PUMP

Important: NEVER CLEAN WITH WATER. You can only clean it with a soft dry cloth.

Clean and sterilize your electric pump before first use and after each use, following the instructions below.

❖ STERILIZATION:

1. **Disassemble the pump.** You can wipe up the pump motor (mechanism) (MU) with a slightly damp cloth and then you must dry it immediately.
2. Wash all other parts with warm soap water and rinse them well. Sterilize **ONLY** the following components of the pump: the silicone insert, the milk container, the silicone valve, the silicone piston. The massage pad should not be heated and sterilized in a microwave oven. The sterilization should not be more than 5 minutes.
3. The silicone valve and silicone insert are made of silicone and should not be sterilized for a long time under high temperature.

❖ RECOMMENDATIONS FOR CLEANING

1. Use a steam sterilizer. Boiling the parts is not recommended. The first reason is that the product will be easily damaged if improper sterilization is applied. Second, this method of sterilization cannot achieve full effect.
2. If you do decide to use boiling as a sterilization method, pour more water into the container so that the parts are completely covered with water. You must not leave the product unattended during the sterilization process. Do not boil for too long - maximum 10 minutes.
3. For breastfeeding mothers who breastfeed every 2-3 hours, we do not recommend sterilizing and cleaning after each use. Excessive disinfection and cleaning can easily lead to premature aging of parts and shorten the life of the breast pump. Thorough cleaning of stains and sterilization once a day is recommended.
4. Store the parts in a clean container until the next use.

IMPORTANT! If you use the breast pump as a means of storing milk, you must take care to disinfect all washable parts of the pump, because otherwise, the milk can easily spoil, as it is a product that is difficult to store.

ATTENTION! Before use, make sure that the pump body, valve, piston and cover are dry.

XI. TROUBLESHOOTING GUIDE

1. Lack of suction:

- Check if you have assembled all parts correctly and if the silicone valve is positioned correctly.
- Ensure that the silicone shield brace is fitted securely to the pump body and a perfect seal is created.
- Check whether there are any damaged parts.
- If you can not turn the pump on, check whether the battery is exhausted. Connect the adapter to the terminal for

charging the motor. Do not use it while charging as it shortens its battery life.

2. No milk is being expressed:

- Check point 1
- Ensure the pump is correctly assembled and that suction is being created.
- Relax and try again.
- We recommend you increase the levels gradually; the most suitable level is the one, with which the suction is painless.

The following is NOT true: the stronger – the better. Too strong suction can hurt the nipple.

3. Pain in the breast area where there is expression:

- If you feel pain and no milk flows during expression, maybe your mammary gland is blocked. This case may result in mastitis. Contact a physician promptly. The complete milk expression will have good effect on mastitis protection.
- You may be using too high level of suction. You do not need to use all the suction that the pump can generate.
- Try to adjust the proper suction and frequency, starting at the first level and gradually increasing the levels.
- Before you start using it, put a warm cloth on your breasts and massage it for about 5 minutes.

4. Flow is too slow:

The speed of milk flow differs for every woman. The higher degree and level of expression do not mean faster flow of milk. This depends on many factors – constitution of the breast, level of possibility, even the mood.

- The more regularly you use the breast pump, the quicker and easier the flow of milk. Before you start using it, put a warm cloth on your breasts and use the massage mode for about 5 minutes before you switch to expression mode.

5. Milk is dropping down the silicone massage pad:

- Stop the pump immediately and clean it well.
- Remove and refit again the silicone cylinder in order to ensure it is firmly attached to the central part of the funnel.
- Try to lean slightly forward.

6. Cracking and discoloration of the pump/ feeding bottle parts:

Avoid contact with antibacterial or abrasive detergents/ cleaners as they can damage the plastic. Combination of detergents, cleaning products, sterilizing solution, softened water and temperature fluctuations can cause plastic to crack. If this occurs, please, do not use. The pump body and the breast milk containers are dishwasher safe but the food colorings may cause discoloring them. Keep the product out of direct sunlight as prolonged exposure may lead to discoloration.

7. Lost or broken parts:

For spare parts contact an authorized service center or the sales agent from whom you have purchased the product. Do not use spare parts, which are not approved or supplied by the manufacturer.

- Do not attempt to repair or to modify the product, because this may endanger Your and Your baby's health.

8. The pump does not work:

- The battery indicator will start blinking if the charge is not sufficient. Charge immediately.
- If even after charging the pump does not work, do not try to fix it yourself. Contact the facility from which you purchased it to have it repaired.

XII. CONFORMITY WITH EUROPEAN STANDARDS AND DIRECTIVES

This product has been designed and manufactured in accordance with the requirements of the European Directives: Directive 2014/35/EU relating to the making available on the market of electrical equipment designed for use within certain voltage limits; the Electromagnetic Coherence Directive 2014/30/EU; Directive 2011/65/EU and its supplements 2015/863/EU on the restriction of the use of certain hazardous substances in electrical and electronic equipment, Regulation (EU) 1935/2004 on materials and articles intended to come into contact with food; Regulation (EU) 10/2011 and its amendments (EU) 2018/213 and (EU) 2020/1245 on plastic materials and articles intended to come into contact with food and in particular the content of BPA (Bisphenol-A).

The product complies with the requirements of the following European standards, some of which are harmonised:

EN 60335-1:2012+A11:2014+A13:2017+A1:2019+A14:2019+A2:2019+A15:2021; EN 62233:2008;
EN IEC 55014-1:2021; EN 55014-2:2021; EN 61000-3-2:2014; IEC62321-2:2021; IEC 62321-3-1:2013; IEC 62321-4:2013+AMD1:2017; IEC 62321-5:2013; IEC 62321-6:2015; IEC 62321-7-1:2015; IEC 62321-7-2:2017; IEC 62321-8:2017.

MADE FOR CANGAROO

Importer and Manufacturer: Moni Trade LTD

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